- Insulated stainless steel cabinet
- Frost resistant down to (-40 °C)
- Additional frontal heating element (-20 °C to -40 °C)
- Fan for ventilation
- 2 x LED lights (meaning defibrillator is visible in the dark)
- Designed to be easy to open, even under conditions of heavy snow and frost.
- Future-proof! (Software can be upgraded)

FUNCTIONAL DESCRIPTION:

- | | |
|---|--|
| If the door of the cabinet is opened... | The SIREN immediately sounds.
<i>(if the door of the cabinet is closed once again within 3 minutes the siren will stop automatically)</i> |
| Once the door is closed... | The cabinet is once again ready for operation. |

TECHNICAL DESCRIPTION:

- | | |
|----------------------|--|
| Door: | The door is fitted with thermopane glass and marked with international AED labels/symbols. |
| Light: | 2 x LED lights in the cabinet, such that the defibrillator is visible in the dark. |
| Siren: | Siren in the cabinet. (100 dBA / 1m) |
| Fan: | Ventilation fan, starts automatically when temperature exceeds +40 °C. |
| Frost-prof cabinet: | Thermostatically-controlled heating element. Frost resistant down to -40 °C |
| Voltage: | Must be connected to a 230V (116Wh) electrical supply. |
| External dimensions: | W x H x D = 44 x 46 x 27 cm (incl. door handles) |
| Internal dimensions: | W x H x D = 33 x 38 x 20 cm |
| Weight: | Gross weight incl. packaging 18.5 kg. |

MAINTENANCE / CLEANING:

We recommend cleaning the outer surface of the cabinet at least 1-2 times per year.
Wipe using detergent and a damp cloth or brush and rinse with clean water. Finally wipe down the cabinet with a dry cloth.

If the surface of the cabinet becomes tarnished, this will be due to deposits which can easily be cleaned off.
For difficult deposits, use a scouring pad. **Never use wire wool.**

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